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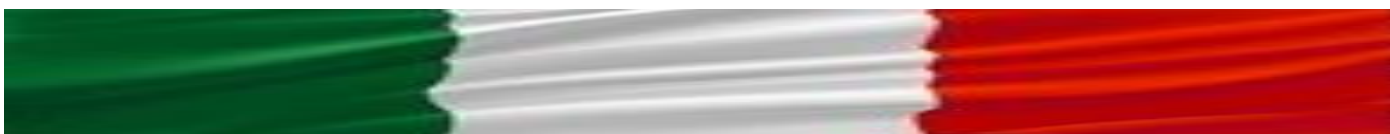
Patient warnings for application, removal and cleaning of the removable individual custom-made device

- 1. Handle carefully to prevent the device from falling.*
- 2. Be sure to insert the upper device into the upper arch, the lower one into the lower arch.*
- 3. Insert the device in situ with two hands.*
- 4. Find the correct position for insertion without excessive pressure.*
- 5. Close your mouth only when the device is perfectly positioned.*
- 6. Disconnect the removable device by following the reverse procedure to that indicated for the application*

First of all, we remind you that aligners can only do their job when you wear them. Your collaboration is therefore essential: you will have to wear the aligner day and night, taking it off ONLY during meals and for oral hygiene.

Generally the aligner you are using after 14 days must be replaced with the next one (unless otherwise indicated by the clinician) as it completes its function. Decreasing the daily hours by increasing the number of days of use of the aligner is NOT a valid alternative, but compromises the possibility to achieve the desired goal.

Once you have finished the aligners in your possession, it is important to continue wearing the last aligner until you receive additional aligners or retention devices from your dentist.

Cleaning of the aligner

Before each use, clean with a toothbrush, cold water and soap.

Never use hot water as the material of which the aligners are made deforms with heat.

Do not use mouthwashes to clean the aligner, they could stain or color it, ruining its aesthetic appearance.

While wearing them we advise you NOT to drink coloring drinks (coffee, tea, red wine, licorice exc.) to avoid staining them.

Aligners should always be worn with clean teeth.